

Steps to use a Tono-pen



A) Calibration : needs to be done once per day

Step 1: Point the tip downward and press the on button twice



Step 2: Wait 15 sec until the screens shows « up »

Step 3: point the tip upward




Step 4: Wait until the screen shows « good », then it is ready for use

If it shows «bad» you need to start over


B) Measurements

Step 1: Place the disposable tip and make sure it is in the correct position

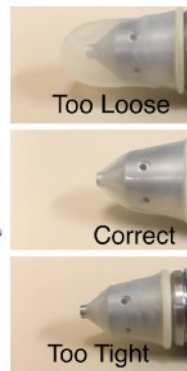

Step 2: Ask the patient to look forward and turn on the Tono-pen.

Step 3: Place the Tono-pen perpendicular to the cornea.

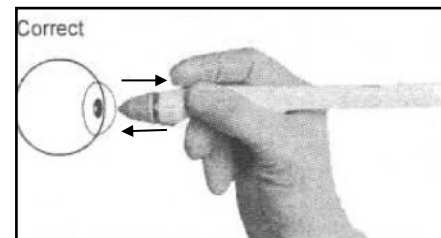

Step 4: Tap gently the surface of the cornea multiple times. You will hear clicks each time.

Step 5: At the final sound, the IOP will appear in the screen

Step 6: Remove the disposable tip and place a new one without pouching the edges.

